



## DESCRIPTION

Elastic ankle with lateral and rear strap. The '8 shape' lateral straps have a remarkable and efficient articular stability and are helpful for patients with tibial or peroneal ligament problems. It is possible to balance the tension on patient need to provide lateral or medial stability.

## CHARACTERISTICS

- SPACER fabric, 100% breathable
- Rear strap for Achilles tendon to unload.
- Ambidextrous.

## INTENDED USE

Therapeutic and not invasive Medical Devices, capable to correct postural problems or capable to stabilize limbs.

## INDICATIONS

Ankle distortions • ankle arthrosis • ankle tendonitis • peroneal tendinopathy • useful for recovering of the agonistic activity

## APPLICATION

1. Wear the brace closing it around the ankle.
2. Cross the straps above the instep (8 shape) and hook them back to each other.

## POPULATION AND USE LIMITATIONS

WEPERE devices are generally indicated for use on adults over 12 years of age. The product should only be used for the intended purpose (see section "Indications").



Hand wash in warm water



Do not tumble-dry



Do not iron



Do not bleach



Product compliant with EU Regulation 745/2017  
MNGP494 of 24/08/2022



I.A.C.E.R. SRL  
Via Enzo Ferrari 2 - 30037 Scorzè (VE) - ITALY  
[www.wepere.com](http://www.wepere.com)



TENORTHO SRL  
Via Trento e Trieste 100 - 20853 Biassono (MB) - ITALY  
[www.tenortho.com](http://www.tenortho.com)

# WEPERE GARD

## ANKLE BRACE WITH "8 SHAPE" STRAPS

REF WE4105



## MAINTENANCE

Hand wash in warm water with mild soap; then rinse it with clean water. Dry away from heat sources. Do not dispose the product in the environment.

## PRECAUTIONS FOR USE

The device must not create pressure on any part of the body that is injured or swollen. Consult your trusted shop or WEPERE if there is any doubt about the application. Do not wear the device near open flames.

## HEALTH SURVEILLANCE

In the interests of continuous health surveillance to ensure the safest possible use of WEPERE products, please report any severe adverse reactions that occur during the use of the product by sending an email to: [quality@iacer.it](mailto:quality@iacer.it)

Also report the event to the competent authorities.

## WARNINGS

The application must be carried out with the utmost care to ensure efficacy, tolerability and correct functioning. In case of any structural or other changes, effectiveness and warranty on the product are not guaranteed. Personal use of the product is recommended. Direct contact of the device with the skin may cause redness or irritation to very sensitive persons. Do not place the product on injured skin. Contact your doctor immediately in case of intolerability.

## DISPOSAL

Do not dispose the product in the environment. Dispose of the product in accordance with the waste laws of your municipality and/or State. The product does not present a risk of contamination.

## WARRANTY

WEPERE certifies that the product has been manufactured correctly, that top quality materials have been used, that all the necessary tests have been carried out and that it complies with the standards and laws in force. WEPERE undertakes to remedy any defect, lack of quality or lack of conformity of the Products imputable to itself, occurring within 6 (six) months from the delivery of the Products and communicated promptly and in any case no later than seven days from the discovery of the defect or fault. WEPERE may choose to repair or replace the Products found to be defective. Products replaced or repaired under warranty shall be subject to the same warranty of 6 (six) months from the repair or replacement. Defects of conformity and, therefore, the legal warranty, do not include any defects or damage caused by accidental events or by the User's responsibility (such as, by way of example but not limited to, carelessness, lack of or incorrect cleaning) or by use of the Products that does not comply with their intended use, or by normal wear and tear. Keep packaging, labels and purchase receipt to ensure proper batch traceability.



Manufacturer



Distributor



Article number



Lot number



Medical Device



Consult the instructions for use



May be used multiple times (multiple procedures) on a single patient



Not been subjected to a sterilization process



Caution using the device