



WEPERE

GARD

WRIST BAND

REF WE1110



DESCRIPTION

Made with breathable elastic fabric. It is useful to control light problems of the lumbar area. Easy to wear, excellent fitting, it offers the right support to the back, giving wellness and relief during daily activity. Thanks to its unique shape, along with its breathability, this brace can be used also for sport activity like water sports, being waterproof.

CHARACTERISTICS

- SPACER fabric, 100% breathable.
- Anterior closure with velcro.
- Posterior straps.
- Lumbar side with band shapeable stays.
- Anterior height 18 cm, posterior 25 cm.

INTENDED USE

Therapeutic and not invasive Medical Devices, capable to correct postural problems.

INDICATIONS

Lumbago • sciatica • mild traumas to lumbar area • muscular contractures • discopathy • spondylarthritis • support during sport and daily activities • alleviation of pain due to a prolonged sitting position in the car • lumbar vertebrae collapse.

APPLICATION

We recommend to wear the guard over a shirt.

1. Unfasten the corset, keeping the thumb loop up.
2. Frontal side made of 3 Velcro (one abdominal, "bigger" and two lateral and "oblique and smaller").
3. Wear the corset and close the abdominal Velcro as needed.
4. First fix the Velcro straps, fixing them properly on the lateral oblique ones.
5. Take the Velcro straps and pull them till reaching the bigger abdominal one, fixing them in the central area.
6. For a perfect fit, assure that the back and lower edge are positioned at 5cm from half of the glutueus.



Hand wash in warm water



Do not tumble-dry



Do not iron



Do not bleach



Product compliant with EU Regulation 745/2017
MNGP494-00 of 24/08/2022



I.A.C.E.R. SRL
Via Enzo Ferrari 2 - 30037 Scorzè (VE) - ITALY
www.wepere.com



TENORTHO SRL
Via Trento e Trieste 100 - 20853 Biassono (MB) - ITALY
www.tenortho.com

POPULATION AND USE LIMITATIONS

WEPERE devices are generally indicated for use on adults over 12 years of age. The product should only be used for the intended purpose (see section "Indications"). Caution is recommended and medical advice should be sought before using lumbar corset in pregnant women. In the same way, particular attention should be paid to patients with pathologies or respiratory difficulties.

MAINTENANCE

Hand wash in warm water with mild soap; then rinse it with clean water. Dry away from heat sources. Do not dispose the product in the environment.

PRECAUTIONS FOR USE

The device must not create pressure on any part of the body that is injured or swollen. Consult your trusted shop or WEPERE if there is any doubt about the application. Do not wear the device near open flames.

HEALTH SURVEILLANCE

In the interests of continuous health surveillance to ensure the safest possible use of WEPERE products, please report any severe adverse reactions that occur during the use of the product by sending an email to: quality@iacer.it

Also report the event to the competent authorities.

WARNINGS

The application must be carried out with the utmost care to ensure efficacy, tolerability and correct functioning. In case of any structural or other changes, effectiveness and warranty on the product are not guaranteed. Personal use of the product is recommended. Direct contact of the device with the skin may cause redness or irritation to very sensitive persons. Do not place the product on injured skin. Contact your doctor immediately in case of intolerance.

DISPOSAL

Do not dispose the product in the environment. Dispose of the product in accordance with the waste laws of your municipality and/or State. The product does not present a risk of contamination.

WARRANTY

WEPERE certifies that the product has been manufactured correctly, that top quality materials have been used, that all the necessary tests have been carried out and that it complies with the standards and laws in force. WEPERE undertakes to remedy any defect, lack of quality or lack of conformity of the Products imputable to itself, occurring within 6 (six) months from the delivery of the Products and communicated promptly and in any case no later than seven days from the discovery of the defect or fault. WEPERE may choose to repair or replace the Products found to be defective. Products replaced or repaired under warranty shall be subject to the same warranty of 6 (six) months from the repair or replacement. Defects of conformity and, therefore, the legal warranty, do not include any defects or damage caused by accidental events or by the User's responsibility (such as, by way of example but not limited to, carelessness, lack of or incorrect cleaning) or by use of the Products that does not comply with their intended use, or by normal wear and tear. Keep packaging, labels and purchase receipt to ensure proper batch traceability.



Manufacturer



Distributor



Article number



Lot number



Medical Device



Consult the instructions for use



May be used multiple times (multiple procedures) on a single patient



Not been subjected to a sterilization process



Caution using the device